**stäubli**  
SWITZERLAND

**KATHETERVENTIL KV 100 EH**  
**VALVE POUR CATHETER KV 100 EH**  
**VALVOLA CATETERICA KV 100 EH**  
**CATHETER VALVE KV 100 EH**

Patent  
Max Stäubli AG, med. Apparatebau, CH-8812 Horgen  
Tel. +41 (0)44 728 80 40, Fax +41 (0)44 728 80 41  
info@maxstaebli.ch, www.maxstaebli.ch

CE 1250

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The catheter valve KV 100 EH is for the use of all patients who are obliged to wear a bladder catheter continuously.

The advantages of the catheter closure system KV 100 EH are as follows:

- The catheter valve KV 100 EH can be operated by one hand. It closes automatically.
- The valve forms a closed system, together with the balloon catheter.
- The valve remains in the catheter outlet and is only removed when the bladder is rinsed. Through this, the danger of infection from the closure side is reduced.
- Because the valve must only be removed when the bladder is rinsed, it sits more firmly in the catheter outlet and does not fall out.
- Urine no longer flows over the hands of the patients or the nursing staff.
- The series of trials has shown that the valve can be operated by nearly all patients. (Even those in an advanced age.)
- The troublesome insertion of the usual closing plug, often carried out with trembling hands, is obviated. (For this reason, more elderly patients often use a urine bag, which can lead to undesirable shrinkage of the bladder.)
- When the valve is opened, the whole diameter of 4 mm is free. The whole of the valve thus presents a free passage and no troubles can arise from blockages.

**Important: The patient must be correctly instructed in the insertion and operation of the valve.**

**Cleaning and disinfection:**

Rinse the open valve with warm water and then place it in a commonly available disinfecting solution but appropriated for synthetic devices, by respecting its relevant instructions.

Should only be applied for one and the same person.