



■ Pantone 5415

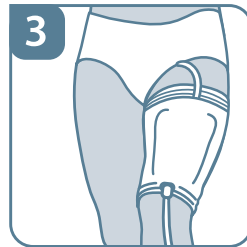

independence  
with confidence

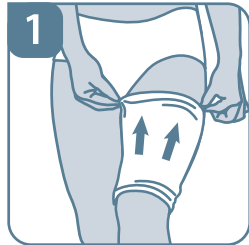

**optimummedical**<sup>®</sup>

Optimum Medical Solutions Ltd  
47 Park Square East, Leeds, LS1 2NL, UK  
Tel.: +44 (0) 8456 435 479  
enquiries@optimummedical.co.uk  
www.optimummedical.co.uk



3005-3006-3007-3008  
#FU-June15-V2.0

## GBR UGO FIX SLEEVE LEG BAG HOLDER

### INSTRUCTIONS FOR USE

It is important that you measure around the widest part of your calf or the widest part of your thigh depending on where you will be wearing your sleeve. Size and colour coding is detailed below.

- 1 Pull the **Ugo Fix Sleeve** over your foot and position on your thigh or calf. Ensure the coloured strip is at the top and facing outwards.
- 2 Slide your leg bag into the front pouch through the opening at the top.
- 3 Push the drainage tap through the hole in the bottom of the **Ugo Fix Sleeve** and adjust to ensure optimum drainage and comfort.

### CARE INSTRUCTIONS



### MATERIAL CONTENT

24% elastane  
76% polyester



### SIZE AND COLOUR CODING

<b>REF 3005</b>	Small	- 24cm – 39cm	- yellow
<b>REF 3006</b>	Medium	- 36cm – 55cm	- blue
<b>REF 3007</b>	Large	- 40cm – 70cm	- brown
<b>REF 3008</b>	Extra large	- 65cm – 90cm	- green

## DEU UGO FIX SLEEVE ARM BEIN BEUTELHALTER

### BEDIENUNGSANLEITUNG

Vor dem Anbringen ist es wichtig, dass Sie um die Wade oder den Oberschenkel messen und die richtige Größe verwenden.

- 1 Ziehen Sie den **Ugo Fix Sleeve** über den Fuß und positionieren Sie ihn an Oberschenkel oder Wade. Stellen Sie sicher, dass der farbige Streifen oben ist und nach außen zeigt.
- 2 Schieben Sie Ihren Beinbeutel durch die Öffnung oben in die vordere Tasche.
- 3 Drücken Sie den Ablasshahn durch das Loch in der Unterseite des **Ugo Fix Sleeve** und stellen Sie diesen ein, um optimale Entwässerung und Komfort zu gewährleisten.

### PFLEGE



### MATERIAL INHALT

24% Elasthan  
76% Polyester



### GRÖSSEN UND FARBCODIERUNG

<b>REF 3005</b>	Klein	- 24cm – 39cm	- gelb
<b>REF 3006</b>	Mittel	- 36cm – 55cm	- blau
<b>REF 3007</b>	Groß	- 40cm – 70cm	- braun
<b>REF 3008</b>	Extra groß	- 65cm – 90cm	- grün