


Offering
three options
for urine drainage
support

1 Securely slots
between mattress
and divan

7

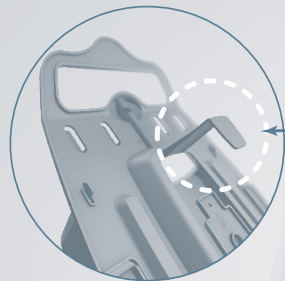

2 Conveniently hooks
onto bed frame
or chair arm

Hook

8



Always ensure the drainage
bag sits below the level of
the bladder.

9

3 Free standing for use in
clinical and residential
settings

10

independence with confidence

Ugo stand
with dignity

designed to support
Ugo 2L Drainage Bags
provided with a **dignity** cover
for discreet protection

REF 3024 CE REUSABLE



optimummedical®

Optimum Medical Solutions Ltd
Floor 1, Unit 5 Armley Court,
Armley Road, Leeds, LS12 2LB, UK
Tel.: +44 (0) 8456 435 479
enquiries@optimummedical.co.uk
www.optimummedical.co.uk



3024-IFU-Apr17-V2.0

Components:



A

Ugo stand™

designed to support
Ugo 2L Drainage Bags




B

**Ugo dignity
cover™**

to be used in conjunction
with your Ugo Stand

2

Assembling the Ugo Stand

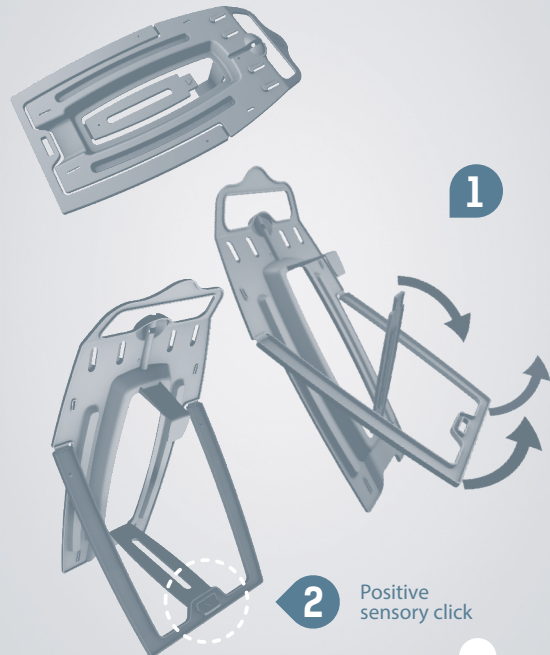

1

2

Positive
sensory click


3

Installing the Ugo 2L Drainage Bag



4

Fitting the Ugo Dignity Cover

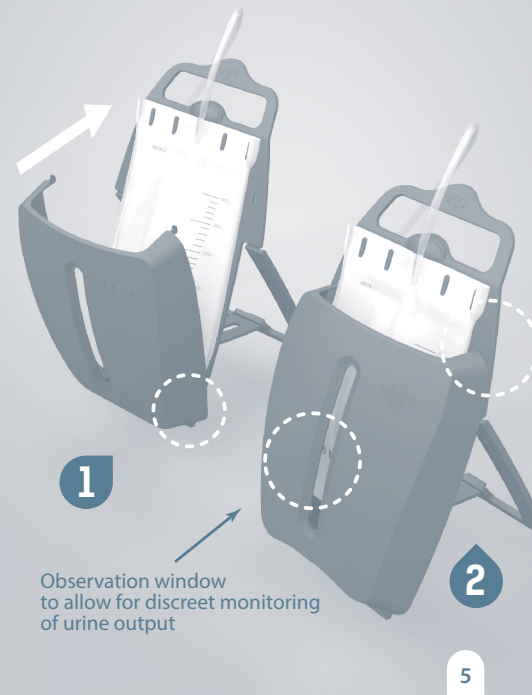


1

2

Observation window
to allow for discreet monitoring
of urine output

5

Removing the Ugo Dignity Cover

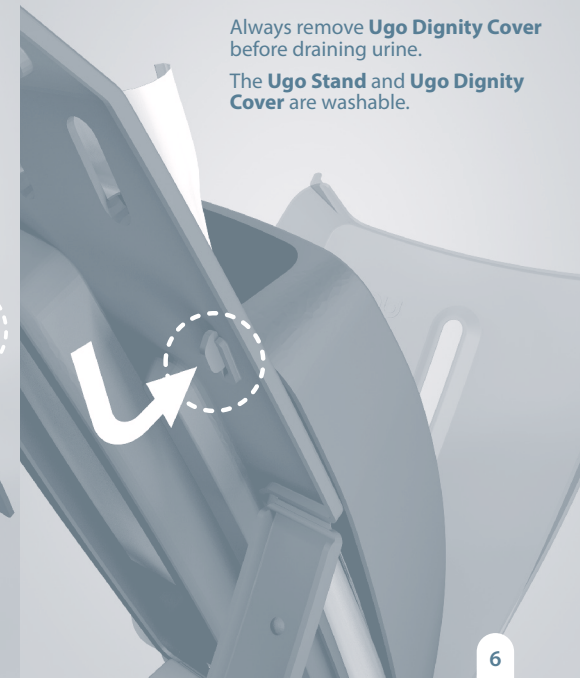

Always remove **Ugo Dignity Cover**
before draining urine.
The **Ugo Stand** and **Ugo Dignity
Cover** are washable.

6