Pantone 2747

GBR

INSTRUCTIONS FOR USE

BRAND: OptiLubereach™

COMPOSITION:

- Glycerine • PEG
- 4-Hydroxybenzoic acid esters
- Sodium hydroxide • Purified water

USE: Use as lubricant.

CONTRAINDICATIONS: Sensitivity to Glycerine.

DIRECTIONS OF USE:

To open the tube - snap and twist the end of the nozzle. Squeeze the body of the tube to release the gel. Discard the tube and any remaining gel after use. For single use only.

RECOMMENDED USE:

OptiLubereach™ Lubricating Jelly is a safe and effective personal and professional lubricant, recommended for:

- Insertion of catheters
- Endoscopy
- Lubrication of condoms
- Rectal thermometers
- General hospitals procedures
- During sexual intercourse
- Enemas/douches
- Gynaecological examination
- Digital & instrument examinations.

OptiLubereach™ is harmless to human tissue, and will not injure rubber, metal appliances, instruments and natural or synthetic textiles. OptiLubereach™ is not a contraceptive.

SIDE EFFECTS AND SPECIAL PRECAUTIONS:

- Precautions:** If product is damaged or seal opened, sterility is not guaranteed.
- Warnings:** If irritation occurs, discontinue use and consult a doctor. Avoid eye contact. Keep out of the reach of children.

IDENTIFICATION: Sterile, transparent, water-soluble jelly, greaseless and light.

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